


	1	2	3	4
	Pos.	Vare nr.	Beskrivelse	Stk.
A	1	1120-3-073	Justerbeslag, voksen Type 2	1
	2	1120-3-088	Joystickbeslag for eu joystick REM 90 op	1
	3	1120-3-092	Bøjle for joystick Europa REM 90	1
	4	CR-00669	Holder for joystick.	1
	5	CR-00920	JOYSTICK, EUROPA 90A	1
	6	E-115-03-065	Hjul samling 4.10/3.50-6/6	4
	7	E-130-1-017	Batteribakke 125-130 modeller	1
	8	S-127-06-102	Sæderør, komplet	1
	9	SR-00009	Vinkel bolt for Rosti	4
	10	SR-00078	Håndgreb M8 x 14	1
	11	SR-00119	10x110 Stålbolt, FZB (8.8)	4
	12	SR-00122	M5 x 16 ch FZB	2
B	13	SR-00128	08 Låsemøtrik FZB	26
	14	SR-00188	M6 x 12 Pinolskrue A2	3
	15	SR-00233	08 Skive A2	28
	16	SR-00288	M5 x 10 ch FZB	2
	17	SR-00318	Gummiprofil 7 x 20 x 90	2
	18	SR-00345	10 Skive A2	4
	19	SR-00432	Trækplade f. håndbremse	1
	20	SR-00509	Gummidæmper	2
	21	SR-00543	Label - Serienummer skilt køretøjer øvrig udland.	1
	22	SR-00573	08 Fjederskive A2	2
	23	SR-00575	10 Låsemøtrik A2	12
	24	SR-00585	M8 x 25 ch A2	1
C	25	SR-00590	06 Fjederskive A2	20
	26	SR-00596	M8 x 30 ch A2	4
	27	SR-00729	M8 x 60 rh A2	4
	28	SR-00732	M8 x 20 rh A2	2
	29	SR-00747	05 Skive A2	2
	30	SR-00802	M6 x 16 ch A2	5
	31	SR-01085	Polyamid skive Ø8 - Ø24 - 2	1
	32	SR-01702	Gummielement, Rosta 38x80	2
	33	SR-01724	25, Rørben SFL Ø25, 4-4,5	2
	34	SR-03071	M5 x 12 ch FZB	2
	35	SR-03178	M5 x 16 rh A2	2
	36	SR-03228	06 Skive A2	4
D	37	SR-03246	10x140 Stålbolt, FZB (8.8)	2
	38	SR-03252	10x50 Stålbolt, FZB (8.8)	6
	39	SR-03396	Batterirør	2
	40	SR-03625	Rørben VL 15x15	2
	41	SR-04130	Motor-servo til MC for joystick	1
	42	SR-04190	M6 x 30 ch A2	15
	43	SR-04342	M8 x 85 ch FZB	4
	44	SR-04558	Dafacell 15 x 25 x 60	2
	45	SR-04580	BREMSEKABEL 320MM, KOMPLET.	2
	46	SR-04772	04,8 x 14,3 RH Popnitte A2	2
	47	SR-05100	Dæmper Stabilus	2
	48	T-127-01-014	Bagplade, komplet.	1
49	T-127-01-016	Bagskærm 130T	1	
50	T-127-01-511	Fodplade 4W inkl. fodmätte	1	
51	T-127-01-550	Skærme afdækning, komplet blå 4w. Joystick.	1	
52	T-127-01-551	Lygtehus 4W. Joystick, komplet.	1	
53	T-127-02-004	Kompl. bagenhed med motor	1	
54	T-127-03-022	Forlygte, komplet	1	
55	T-127-03-510	Forbro med affjedring, komplet (4W)	1	
56	T-127-03-551	Styrestang, 4W (joystick)	2	
57	T-127-05-550	Elektronikplade komplet for Joystick styring.	1	
58	T-127-1-005	Inderskærm højre.	1	
59	T-127-1-006	Inderskærm venstre.	1	
60	T-127-1-024	Gummiprofil 3 x 25 x 8 x 480	1	
61	T-127-2-043	Bagbom	1	
62	T-127-2-055	Holder for fodplade	1	
63	T-127-2-060	Beslag for dæmper, højre.	1	
64	T-127-2-065	Beslag for dæmper, venstre.	1	
65	T-127-2-550	Bundramme 4W. Joystickstyring	1	
66	T-127-3-004	Kapsel for fælg alu 8"	2	
67	T-127-3-520	Forkofanger. 4W	1	
68	SR-00293	05-15, Skærmskive A2	5	
69	SR-03178	M5 x 16 rh A2	2	
70	SR-03435	04,8 x 10,9 RH Popnitte ALU/STÅ	6	
71	SR-00678	Afstandsstykke, HDPE Ø20/5 x 3	3	
72	SR-00697	M5 x 20 rh A2	3	

5	6	7	8
NO.	REVISION		DATE
A	New		07-04-2005
			Rev.By. j.meldhede

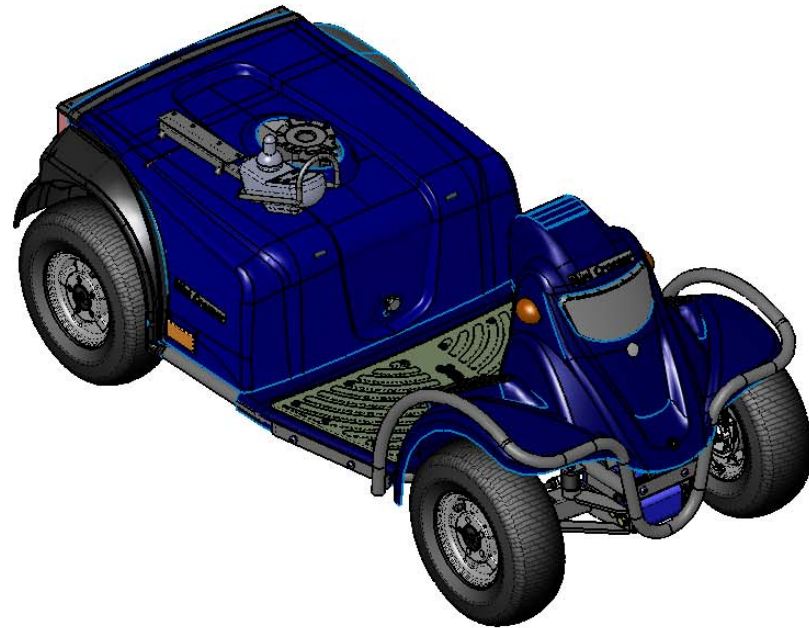


Weight (kg):	56	Surface treatment:	
General tolerances:	ISO 2768-1-m	Geometrical tolerances:	ISO 2768-2-L
		Welding tolerances:	EN/ISO 13920-AE
		Min. general roughness: Ra	3.2
		Material:	
Mini Crosser A/S		Dimensions in mm	European projection
Engsgaardvej 7, DK-7400 Herning Tlf. +45 70 10 20 54 Fax. +45 97 16 85 82		Beskrivelse:	Scale: 1:9
Internet: http://www.minicrosser.com		Scooter T-127 Samling, Joystick.	Sign: j.meldhede
		Description:	Date: 07-04-2005
		Scooter T-127 Assembly, Joystick.	Dwg.no. Issue
			T-127-0-500 A

The information contained in this drawing is the property of Minicrosser A/S. Any reproduction in part or as a whole without the written confirmation of Minicrosser A/S is prohibited.

Beskrivelse: Scooter T-127 Samling Joystick.

Description: Scooter T-127 Assembly Joystick.

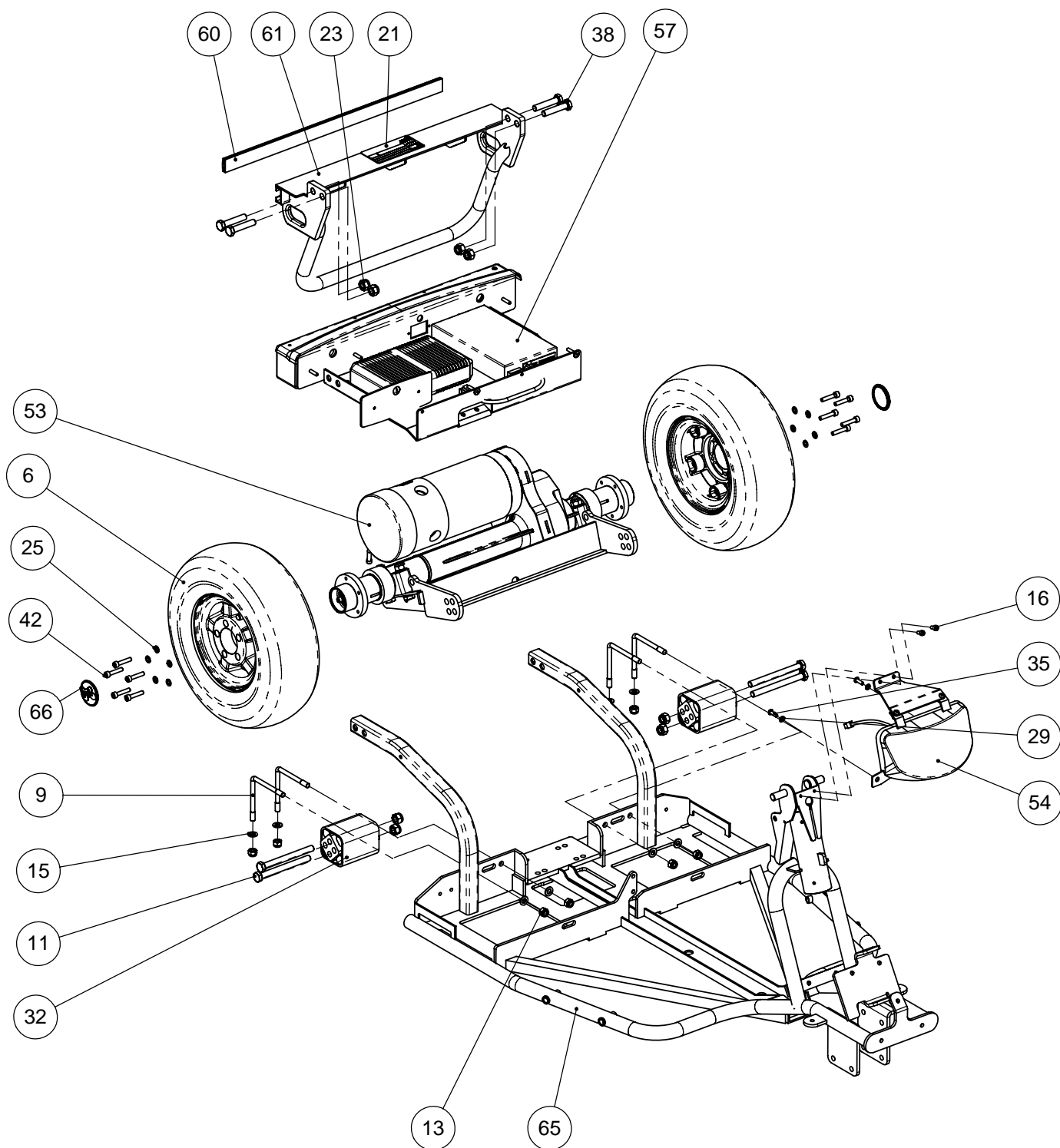

Dwg. no. **T-127-0-650**

Issue:

A

Rev. Date:


07-04-2005

Mini Crosser A/S

SHEET 2 OF 7

Beskrivelse: Scooter T-127 Samling. Joystick.

Description: Scooter T-127 Assembly Joystick.

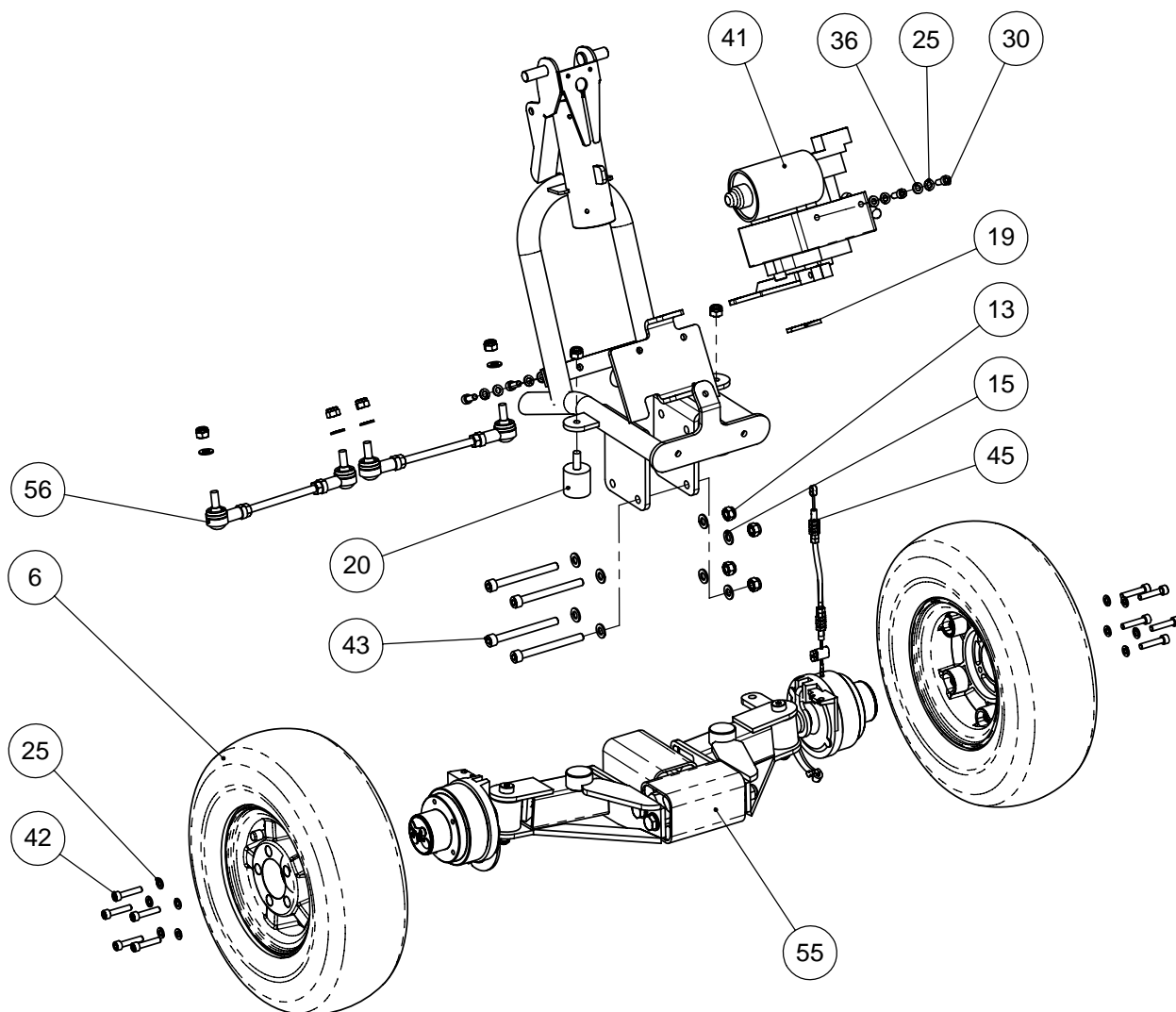

Dwg. no.
T-127-0-650

Issue:
A


Rev. Date:
07-04-2005

Mini Crosser A/S

SHEET 3 OF 7

Beskrivelse: Scooter T-127 samling Joystick.

Description: Scooter T-127 assembly Joystick.

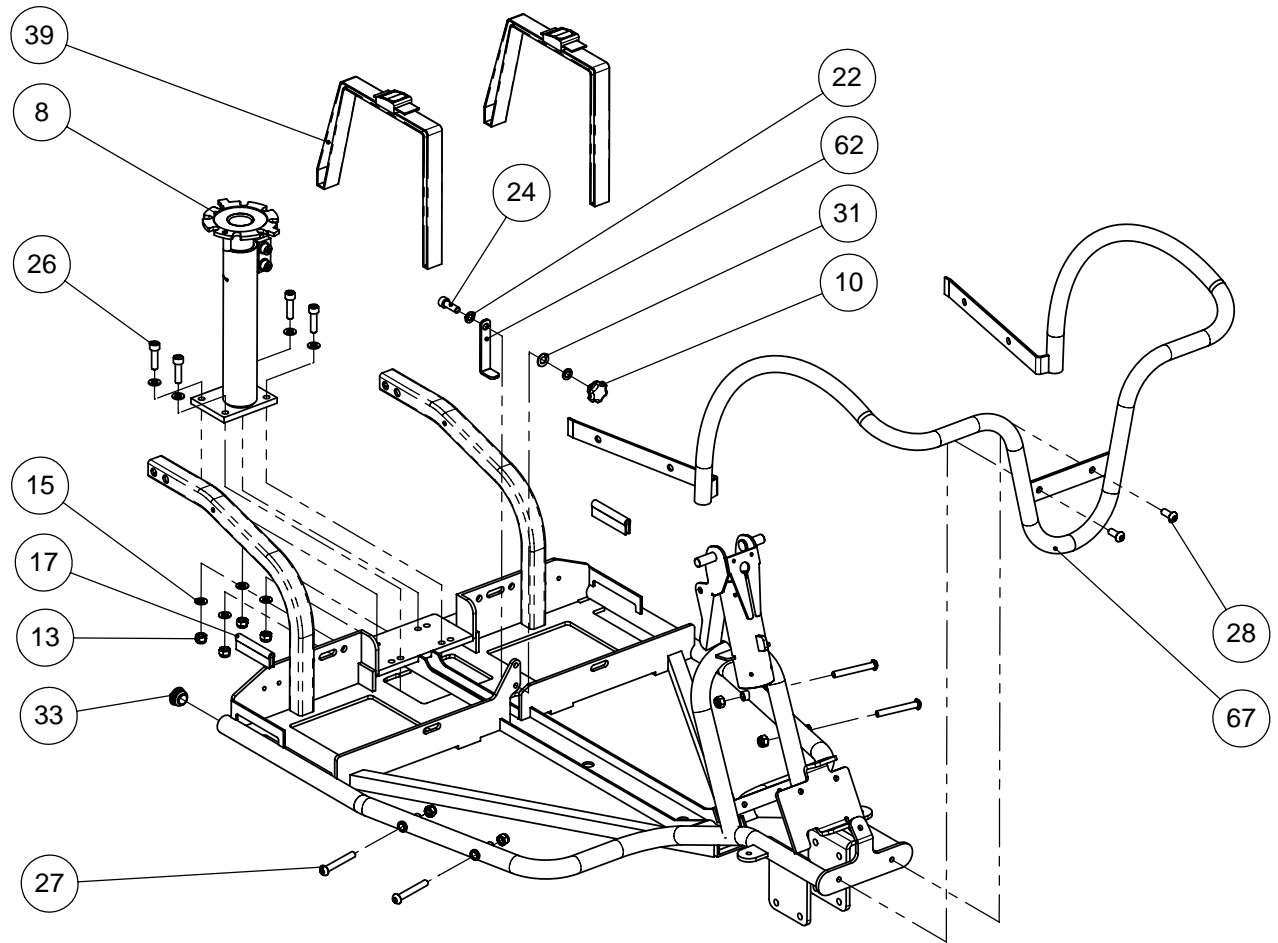

Dwg. no.
T-127-0-650

Issue:
A


Rev. Date:
07-04-2005

Mini Crosser A/S

SHEET 4 OF 7

Beskrivelse: Scooter T-127 samling Joystick.

Description: Scooter T-127 assembly Joystick.

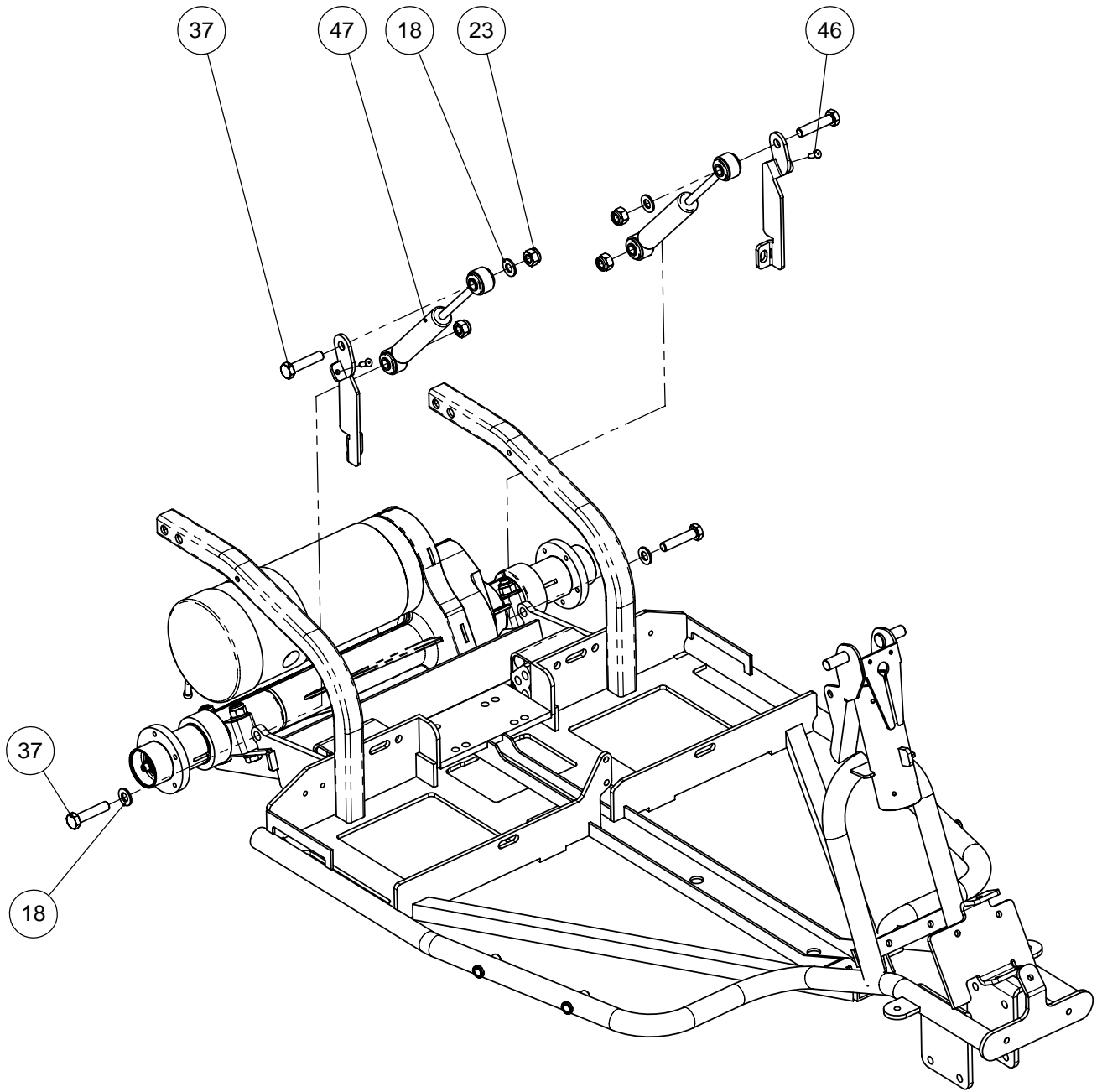

Dwg. no.
T-127-0-650

Issue:
A


Rev. Date:
07-04-2005

Mini Crosser A/S

SHEET 5 OF 7

Beskrivelse: Scooter T-127 Samling. Joystick.

Description: Scooter T-127 Assembly. Joystick.

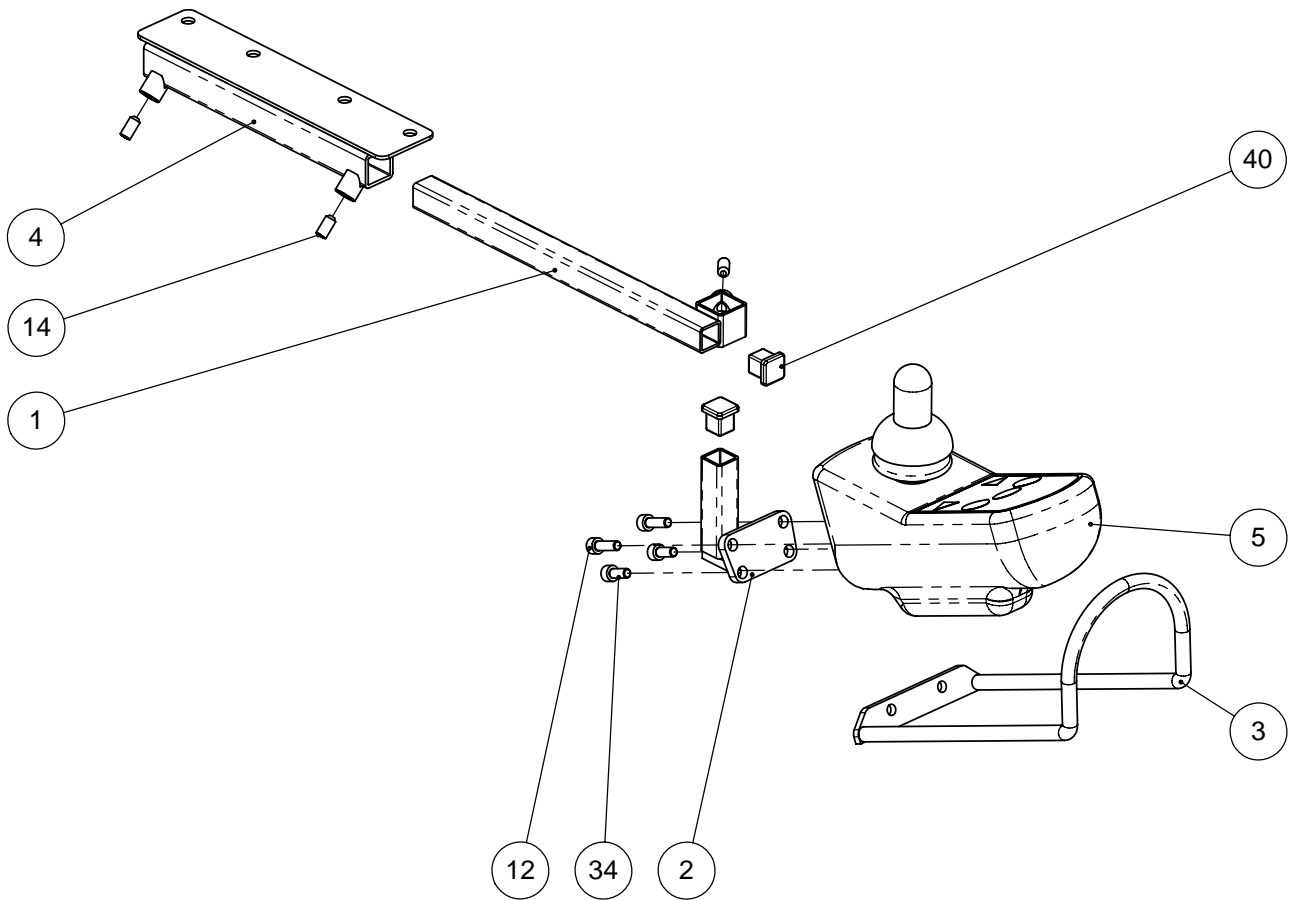

Dwg. no.
T-127-0-650

Issue:
A


Rev. Date:
07-04-2005

Mini Crosser A/S

SHEET 6 OF 7

Beskrivelse: Scooter T-127 Samling. Joystick.

Description: Scooter T-127 Assembly. Joystick.

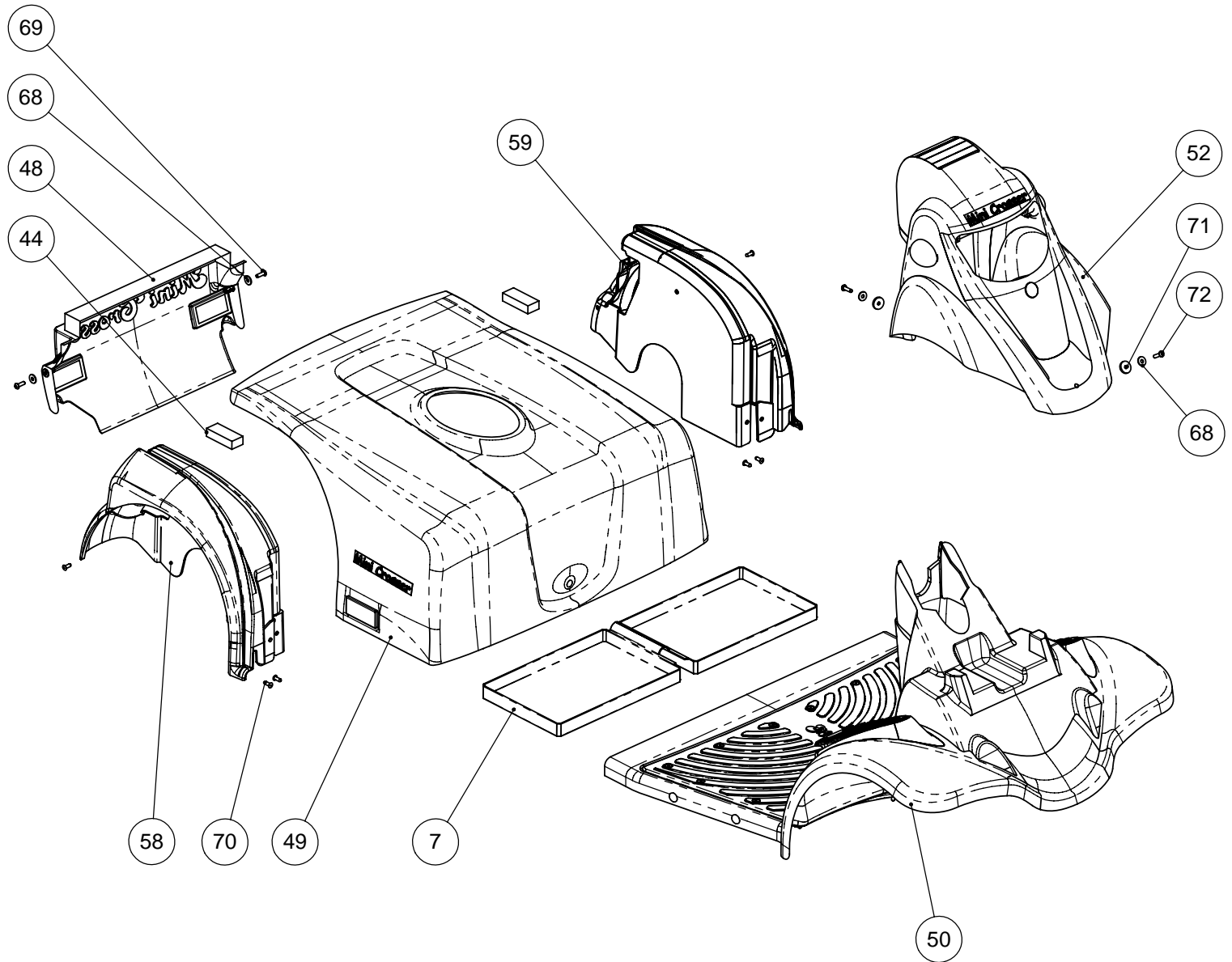

Dwg. no.
T-127-0-650

Issue:
A

Rev. Date:
07-04-2005

Mini Crosser A/S

SHEET 7 OF 7