
Vedr.: M2/X2 software revision 32	PUP nr.:	20-010
	Udarbejdet af:	KSM
	Dato:	11-08-2020
	Intern <input type="checkbox"/>	Ekstern <input type="checkbox"/>

Fra og med scooter med serienummer 03-49962-20-08 benyttes software revision 01.00.0032 i brugerpanelet.

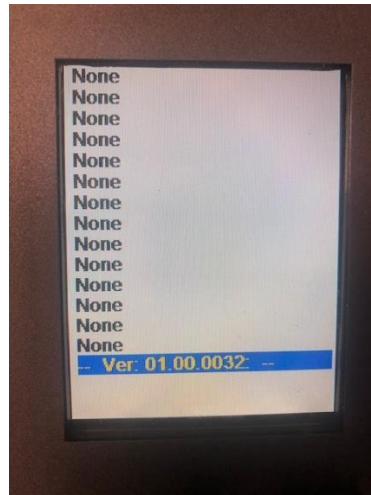Punkt 1:

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
Klaus Smedegaard
R & D Manager

A handwritten signature in blue ink that reads "Klaus Smedegaard".

Regarding: M2/X2 software revision 32	PUP No.:	20-010
	Prepared by:	KSM
	Date:	11-08-2020
	Internal <input checked="" type="checkbox"/>	External <input checked="" type="checkbox"/>

From and included serial number 03-49962-20-08 the software revision 01.00.0032 is installed in the user panel.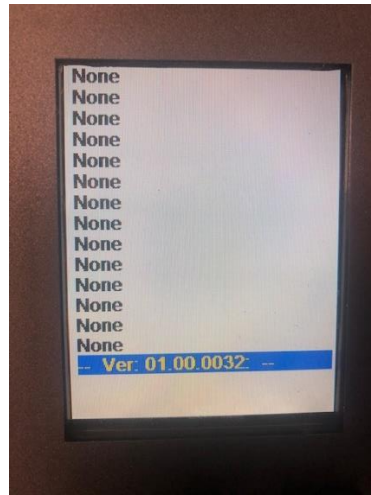

The change in the new software is that the scroll speed is increased when you go up and down in the service menu by pressing the plus or minus button.

Another change is that earlier when you did go up and down in the menu fast you could risk that you would jump out of the service menu. You would see the lines fading out, the screen going blank and jump out to the user interface.

This is now also solved.

Point 1:

The scroll speed has been increased when you go up and down in the service menu. If you hold down the plus or minus button you will see the speed is significant faster.

Point 2:

We have introduced the possibility of jumping one full page in the service menu.


If you press one time on the minus button to indicate the direction down and there after presses the light button you will jump one full page every time you press the light button in the down ward direction here after.

If you press one time on the plus button to indicate the direction up and there after presses the light button you will jump one full page every time you press the light button in the upward direction.

The light button will not turn on and off the lights when doing so when you are in the service menu.

Jumps one full page each time you press the light button.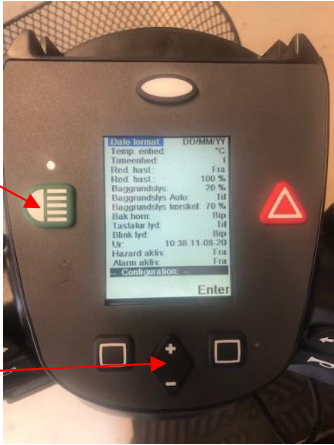

Indicate direction

Point 3:

Earlier the service menu could risk fading out if you scrolled too fast with the +/- button. This was noticed by the lines would disappear, the screen would go white and go back to the user menu. This is now also solved.

Klaus Smedegaard
R & D Manager