

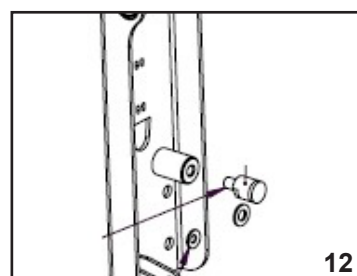
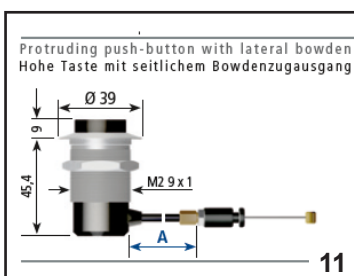
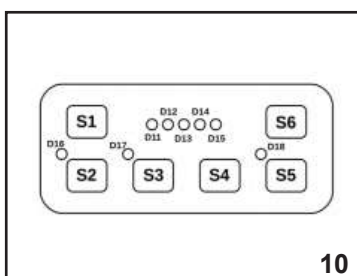


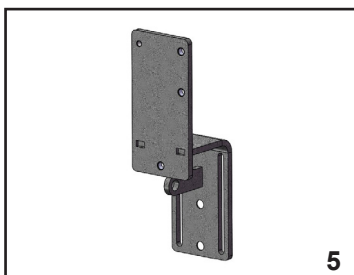

# Mo-vis Scoot control

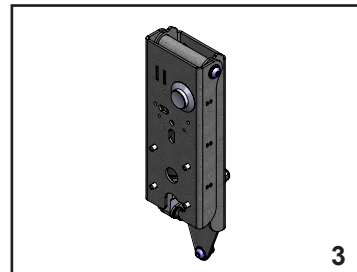

# Mo-vis Scoot control

## Mo-vis Ledsagerstyring Mo-vis Scoot control

Pos.	Del/part no.	Benævnelse	Part name	Antal/ qty.
1	2419-1026	Movis Ledsagerstyring Intelligent R-Net inkl. beslag	Movis Scoot control Intelligent R-Net incl. bracket.	1
2	2419-1024	Movis Ledsagerstyring, R-Net, Intelligent	Movis Scoot control, R-Net Intelligent.	1
3	2419-1025	Beslag for Movis Ledsagerstyring, Intelligent, R-Net	Bracket for Movis Scoot control, Intelligent, R-NET.	1
4	2405-1454	Knapboks for R-Net CJSM2, 5-funktioner	Control box for R-Net CJSM2, 5 functions.	1
5	2406-1597	Rygbeslag for Movis Ledsagerstyring Spinalus-1, Barn/Junior	Back bracket for Movis Scoot control Spinalus-1, child/junior.	1
6	2406-1654	Forhøjer for Movis Ledsagerstyring, komplet	Extension for Movis Scoot control, complete	1
7	2419-1060	Ledsagerstyring gas højre	Throttle right for Movis Scoot control	1
8	2419-1061	Ledsagerstyring gas venstre	Throttle left for Movis Scoot control	1
9	2419-1062	Ledsagerstyring main housing	Main housing for Movis Scoot control	1
10	2419-1063	Ledsagerstyring knap-panel	Control panel for Movis Scoot control	1
11	2419-1037	Trykknop for gasdæmper (bowdentræk), Movis	Push button for gas damper (bowden pull) Movis	1
12	8031-1375	Magnet GN 52.5-ND-16-M8, Movis	Magnet GN 52.5-ND-16-M8, Movis	1



**medemagroup**