



dynamic pacer.

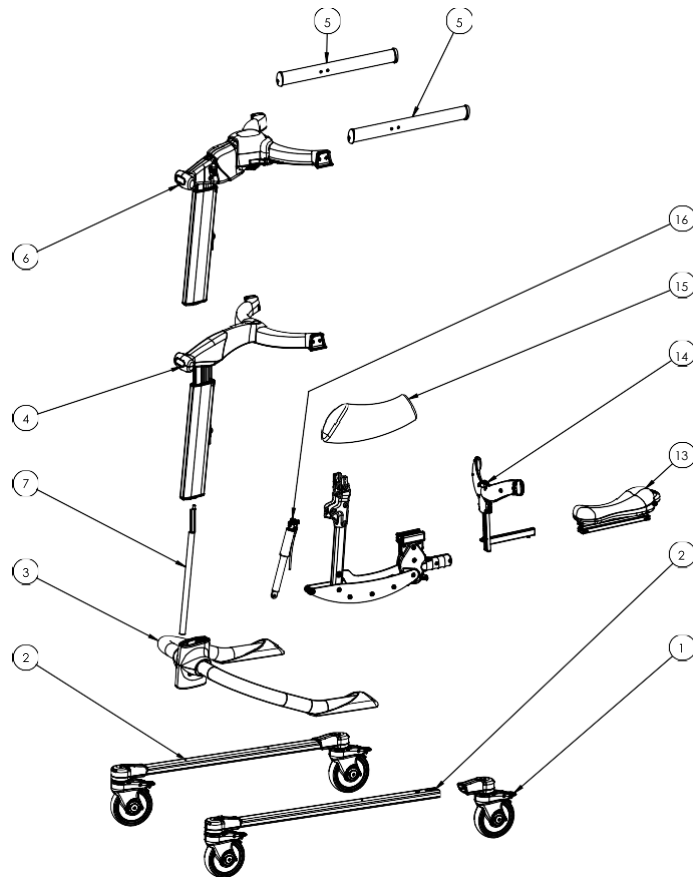

dynamic pacer. size XL

Pos	Item no	Description
1	1303-1060	wheels for standard base
2	1302-1081	main frame left or right standard base size XL
3	1302-1082	main frame upper part standard base size XL
4	1302-1101	central part standard upper frame
5	1302-1084	left or right side upper frame size XL
6	1302-1085	central part dynamic upper frame size XL
7	1302-1111	gasspring upper frame for size XL
13	1321-1002	upholstery seat MPS size M-XL (sadel)
14	1322-1001	cover MPS size M-XL
15	1313-1077	hip support MPS L-XL
16	1302-1060	gasspring MPS for size M-L


dynamic pacer.

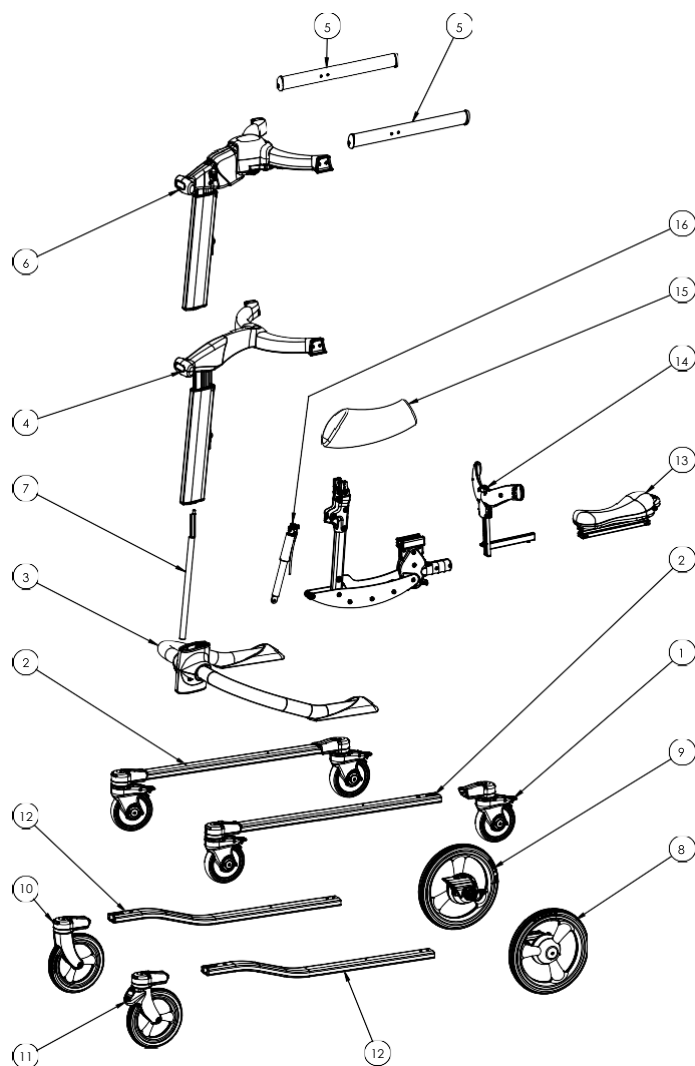

dynamic pacer. size L

Pos	Item no	Description
1	1303-1060	wheels for standard base
2	1302-1087	main frame left or right standard base size L
3	1302-1088	main frame upper part standard base size L
4	1302-1101	central part standard upper frame
5	1302-1090	left or right side upper frame size L
6	1302-1091	central part dynamic upper frame size L
7	1302-1110	gasspring upper frame for size L
8+9	1303-1090	Rear wheels for utility base
10+11	1303-1097	Front wheels for utility base
12	1302-1092	main frame left or right utility base size L
13	1321-1002	upholstery seat MPS size M-XL (sadel)
14	1313-1079	hip support MPS L-XL
15	1322-1001	cover MPS size M-XL
16	1302-1060	gasspring MPS for size M-L


dynamic pacer.

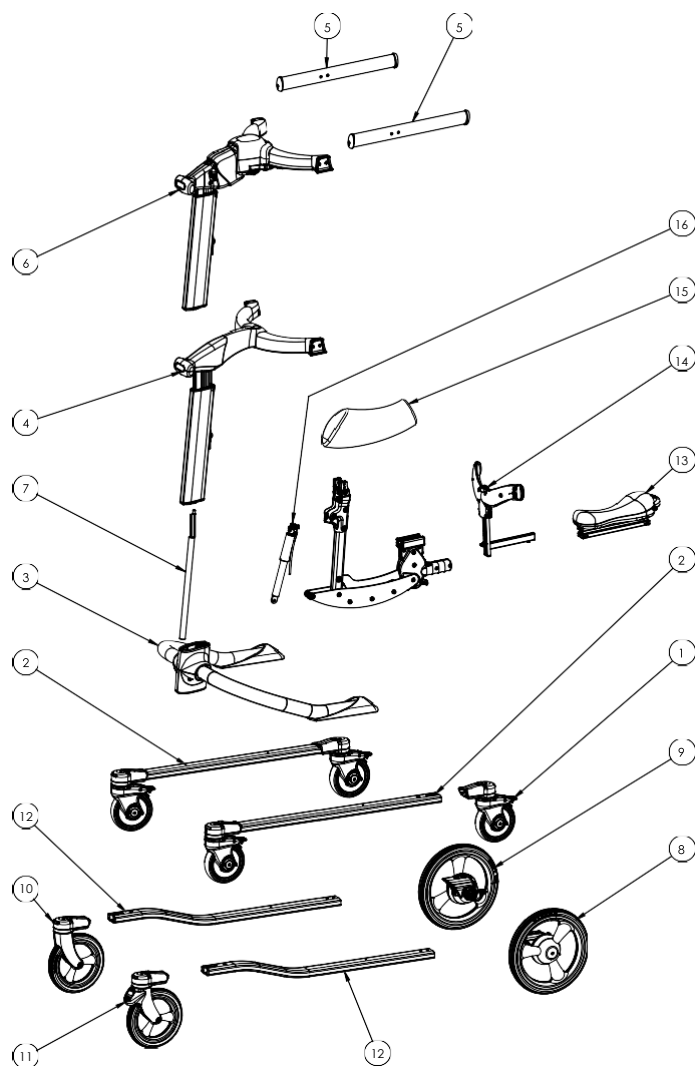

dynamic pacer. size M

Pos	Item no	Description
1	1303-1060	wheels for standard base
2	1302-1093	main frame left or right standard base size M
3	1302-1094	main frame upper part standard base size M
4	1302-1101	central part standard upper frame
5	1302-1096	left or right side upper frame size M
6	1302-1097	central part dynamic upper frame size M
7	1302-1082	gasspring 100N upper frame for size M
8+9	1303-1090	Rear wheels for utility base
10+11	1303-1097	Front wheels for utility base
12	1302-1098	main frame left or right utility base size M
13	1321-1002	upholstery seat MPS size M-XL (sadel)
14	1313-1078	hip support MPS M
15	1322-1001	cover MPS size M-XL
16	1302-1060	gasspring MPS for size M-L


dynamic pacer.

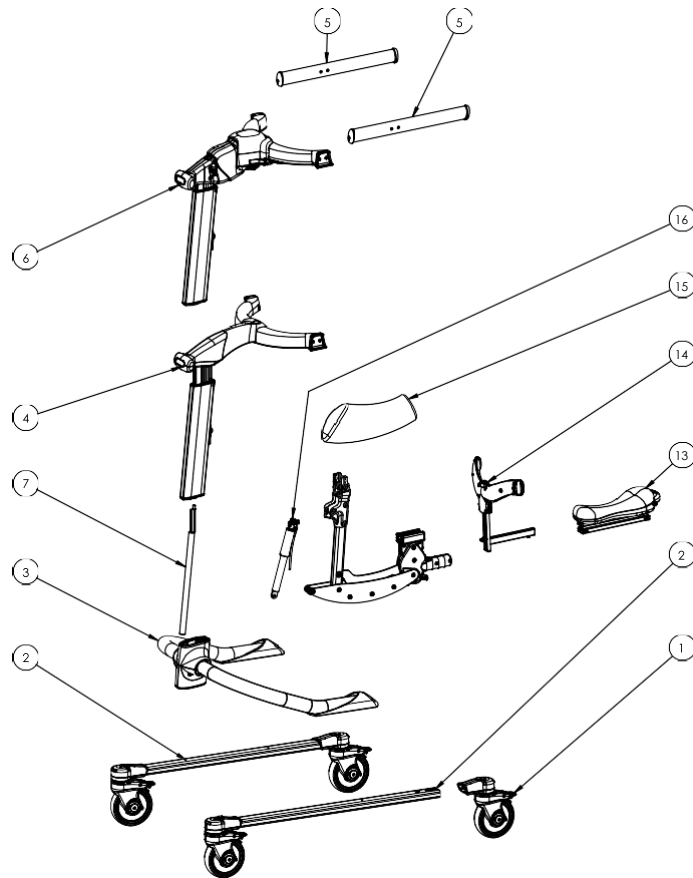

dynamic pacer. size S

Pos	Item no	Description
1	1303-1122	wheel Pacer small +Mini
2	1302-1099	main frame left or right standard base size S
3	1302-1100	main frame upper part standard base size S
4	1302-1101	central part standard upper frame
5	1302-1102	left or right side upper frame size S
6	1302-1103	central part dynamic upper frame size S
7	1302-1044	gasspring 100N upper frame for size S
13	1307-1031	upholstery seat MPS size S (sadel)
14	1313-1078	hip support MPS size 2
15	1322-1002	cover MPS size S
16	1302-1080	gasspring MPS for size S

dynamic pacer.

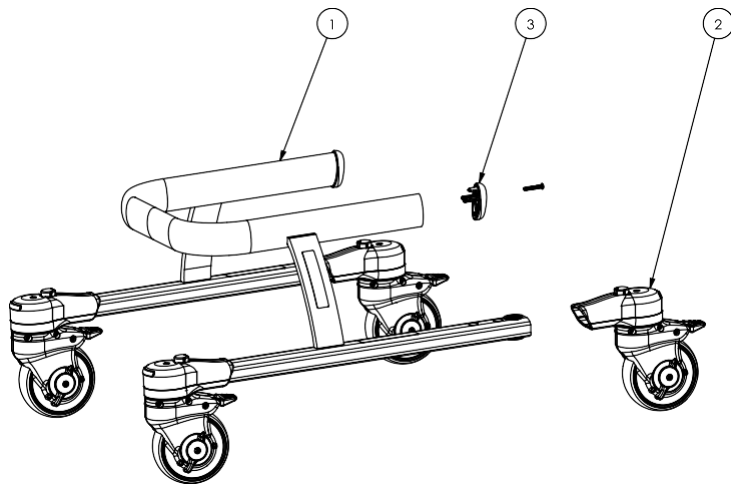

dynamic pacer. size mini		
Pos	Item no	Description
1	1302-1105	Frame Pacer mini
2	1303-1122	Wheel for Pacer mini incl. Bracket
3	1301-1001	Plastic cap