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**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

**1.0 Foreword:**

Dear customer, we are pleased that you have chosen a quality product from *rehastage GmbH*. A good relationship with our customers is of great importance for *rehastage*. Customer proximity means: fast service, as little bureaucracy as possible and close cooperation with our customers. Seek advice from your attending physician or therapist to make sure this product is right for you and meets your needs.

**⚠ For your personal safety, please read these instructions for use carefully before using the product. If there are comprehension problems with the information, warnings and instructions listed here, please inquire with your specialist dealer to avoid any injuries or property damage caused by incorrect operation.**

Do not throw away these instructions for use even after assembling the aid so that you can use them later if necessary.

If you need spare parts or accessories or simply have a question about your aid, we are there for you. We want you to be satisfied with our products and service. We at *rehastage* are therefore constantly working on the further development of our products. For this reason, changes in form, technology and equipment may occur in our product range. Therefore, no claims can be derived from the information and illustrations in these instructions for use.

**2.0 Intended use:**

The floor-ceiling bar "Mister X" is intended as a straightening and standing aid when using the sink and toilet, as well as an entry and exit aid when using bath, shower, or bed. The aim of the supply is to enable independent or more independent personal hygiene or the independent or more independent use of toilets. Furthermore, the floor-ceiling bar "Pole" can also be used in other situations in which appropriate support is required when erecting or standing.

- Indication: The floor-ceiling bar "Mister X" serves patients
  - with dysfunction of the lower extremities due to limited or completely suspended mobility of the joints,
  - with restriction of the muscular strength of the lower extremities,
  - in case of damage to the lumbar spine with impairment of joint mobility with sufficient functionality of the upper extremities and sufficient trunk stability,
  - in the case of generally reduced or impaired mobility without age restrictions.

- Contraindications:

**⚠** Additional safety measures should be taken for persons belonging to certain risk groups. This includes persons with a mental impairment (including persons under the influence of medication), very large persons or persons with severe obesity, as well as any condition that impairs a person's ability to use the floor-ceiling bar "Mister X" without assistance.

**⚠** If you have an illness or injury, or are taking medications that affect your sense of balance or your ability to stand or sit, please consult your physician before using this product. Improper use of the product can lead to serious injury or damage.

**⚠** The product is not intended to carry the full weight of a person. This product is only used as an equilibrium aid and support when standing, straightening up and sitting down.



The floor-ceiling bar "Mister X" is a class I medical device according to Regulation (EU) 2017/745 of the European Parliament on medical devices.

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

**3.0 Scope of delivery:**

Part.	Number	Description
A	1	Blanket bracket
B	2	Extension tube
C	1	Adjustment device
D	1	Base plate (foot)
E	2	Mounting tube
F	1	Threaded protective cover
G	1	Adjustment wrench
H	1	Allen key
J	2	Clip for height adjustment
K	5	Screws

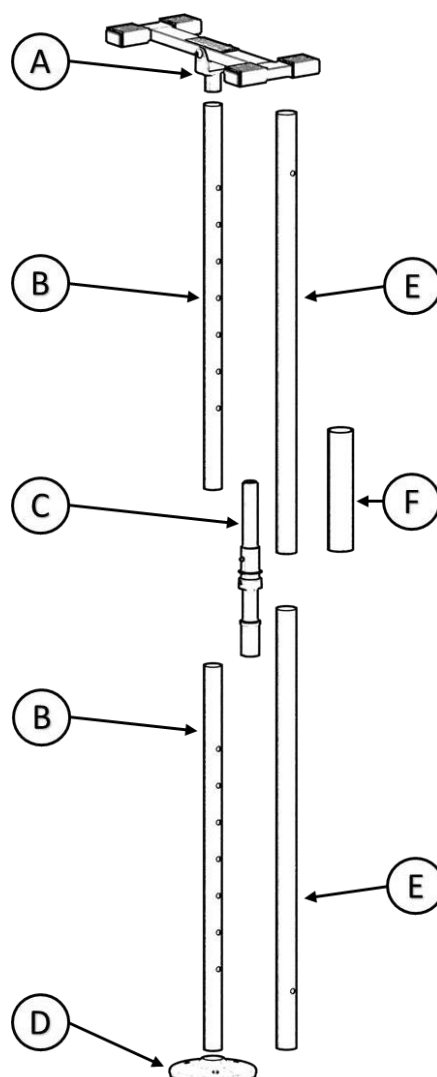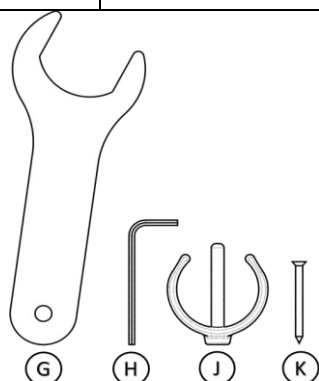**Figure 1**

**4.0 Assembly instruction**

The floor ceiling bar "Mister X" is designed for ceiling heights from min. 213 to max. 305 cm. Using the optional extension tube MRX-7625-xx, the floor and ceiling bar "Mister X" can be used up to a ceiling height of 370cm

Before starting installation, determine where you want to use this product and check whether the given ceiling height and the local conditions are suitable for the installation of the floor-ceiling bar. Please note the following points!

⚠ When choosing the installation location, make sure that a minimum distance of 32 cm is maintained between the floor, ceiling bar and walls or fixed installations such as toilets, bath and shower trays, beds, etc. Under certain circumstances, falling below this distance can lead to entrapment and thus to injury to the user.

⚠ The floor and ceiling bar must not be installed in a shower or bathtub.

⚠ **Not suitable for clamping the floor-ceiling bar are:**

- suspended ceilings,
- Ceilings in drywall design (e.g., plasterboard)
- Paneled or clad ceilings.


**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

Due to the generally weak pressure stability of these ceiling variants, it is not possible to achieve a sufficient clamping effect for a safe installation of the floor-ceiling bar. Furthermore, the clamping process of the floor-ceiling bar with insufficient pressure stability of the ceiling construction can lead to damage or permanent damage in the form of impressions, dents or breaks on or in the ceiling.

- ⚠ It is recommended to carry out the assembly with 2 people to avoid possible personal injury or damage to the ground or surrounding objects.
- ⚠ Excessive movements or inclinations of the product may cause the **ceiling retaining bracket (A)** to come loose during assembly, potentially causing injury or damage to the floor or surrounding objects.
- ⚠ After the rod is fixed between the floor and ceiling, check the length of the exposed threaded surface. The ceiling floor bar must not be used, if the total length of the exposed thread sections plus adjusting ring on the **adjustment device (C)** is more than 17 centimeters.

4.1 Insert the **adjustment device (C)** into the **mounting tubes (E)** with the thread facing the ceiling as shown in **Figure 2**.

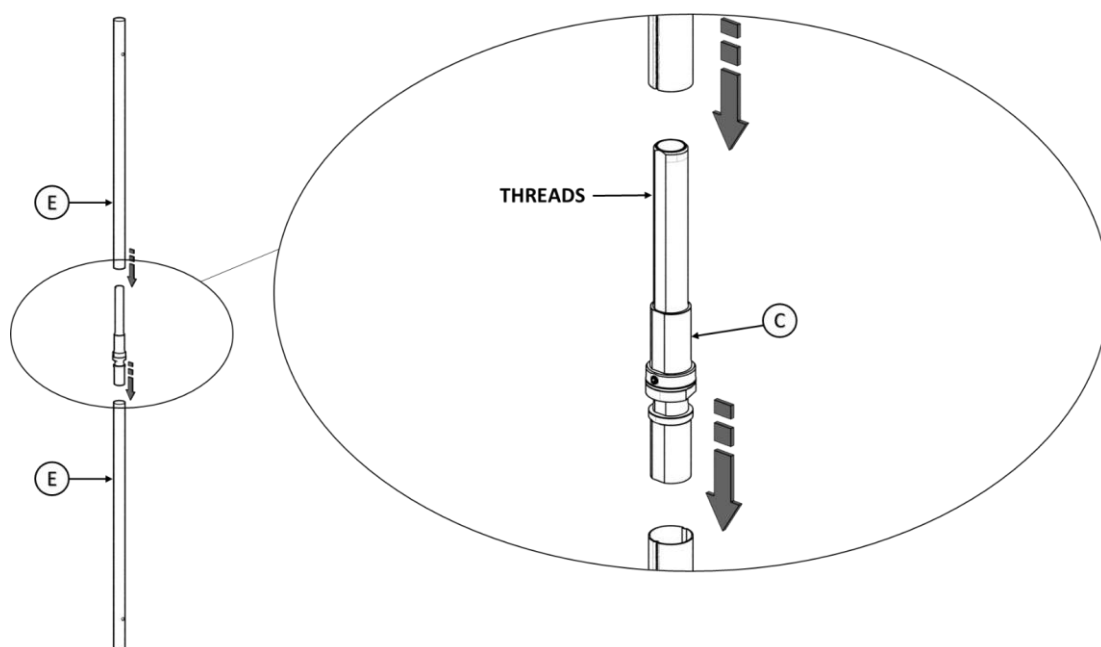
**Figure 2**

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

4.2 Determine the height between floor and ceiling and use the diagram in **Figure 3** to determine the appropriate hole in the **extension tubes (B)** and proceed to **step 4.3**.

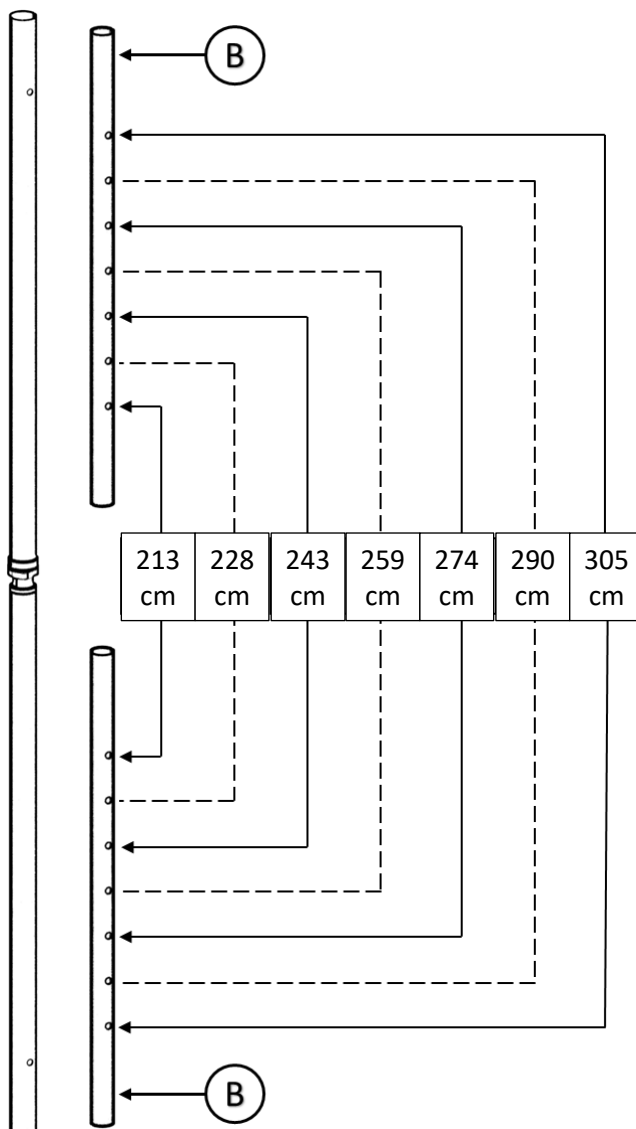
**FIGURE 3**

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

4.3 Insert the **extension tubes (B)** into the **mounting tubes (E)** and align the corresponding holes in the **extension tubes (B)** with the holes in the **mounting tubes (E)**, as shown in Figure 4. Fix the specified pipe position by mounting the **clips (J)**.


⚠ Take care that the **clips (J)** are form-fitting to the pipe.

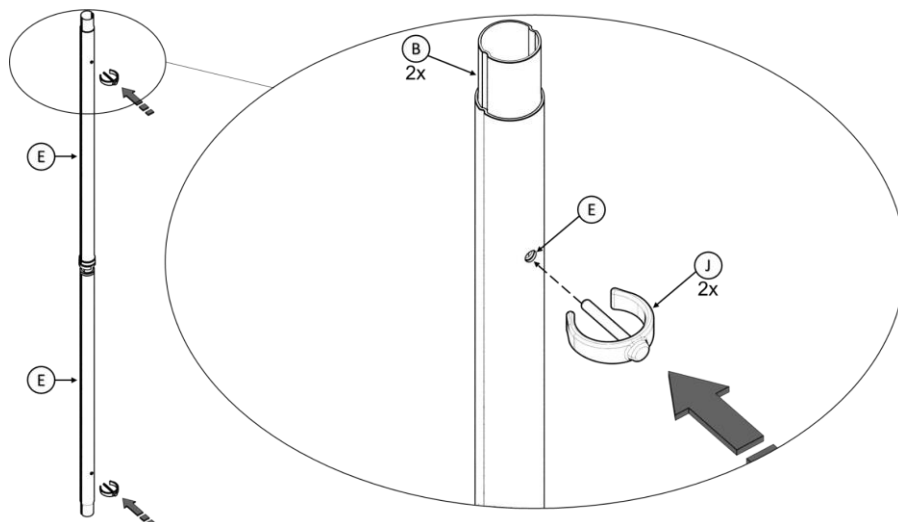**Figure 4**

4.4 Place **base plate (D)** on the floor at the designated location. Insert the **ceiling bracket (A)** into the top **extension tube (B)** and insert the **lower extension tube (B)** into the **base plate (D)** as shown in Figure 5.

⚠ Make sure that the **base plate (D)** rests fully and flat on the floor!

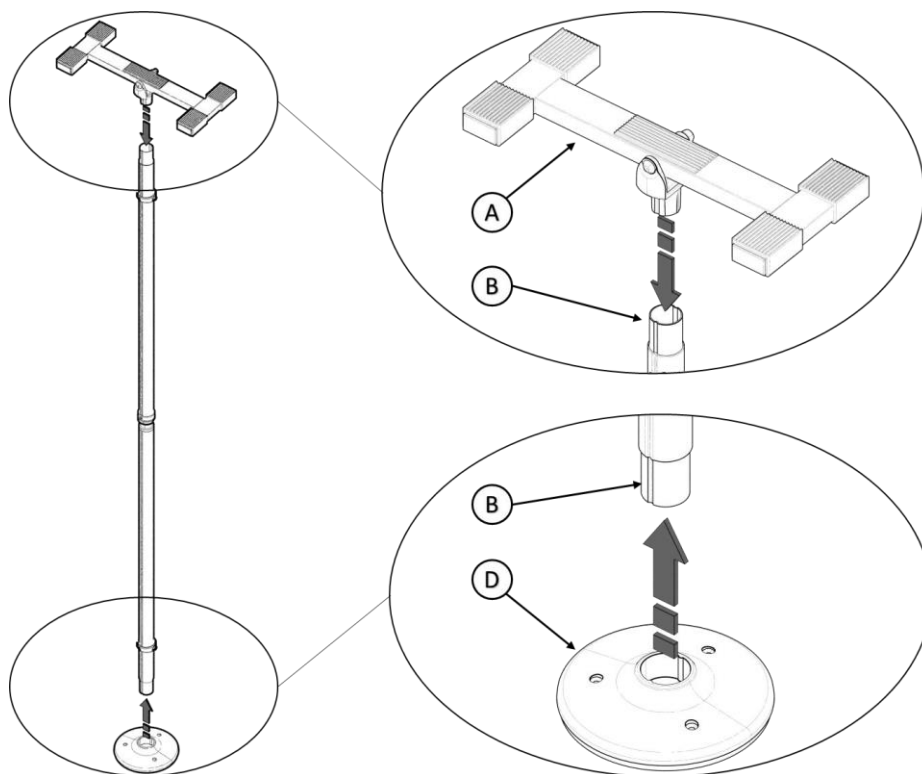
**Figure 5**

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

⚠ Before executing point 4.5, make sure that the floor and ceiling bar is perpendicular and that the **ceiling bracket (A)** is in a parallel position to the ceiling.


4.5 Locate the adjustment nut of the **adjustment device (C)** with the **adjustment wrench (G)**, as shown in **Figure 6**, until the product is fixed between the floor and ceiling without any pressure or slippage. Tighten the **maggot screw** in the **adjustment ring** of the **adjustment device (C)** with the **Allen key (H)** and cover the area with **thread cover (F)**.

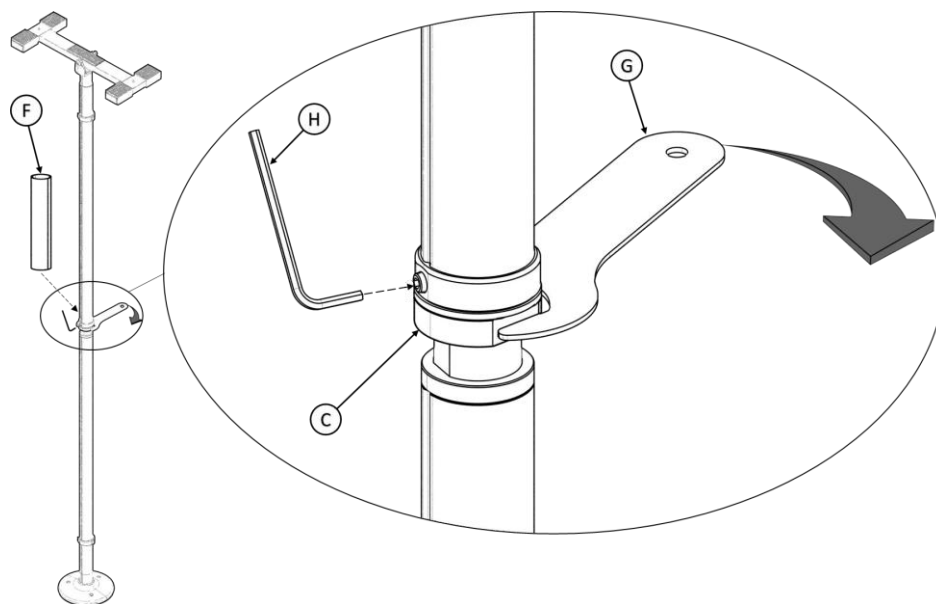**Figure 6**

4.6 Installation with sloping ceiling (sloping roof) If the room corner is inclined in relation to the floor (e.g., slight sloping roof), remove the locking element from the ceiling **retaining bracket (A)**. Use a 6 mm Allen key (not supplied) and a 16 mm wrench (not supplied) to loosen the screw connection on the **ceiling bracket (A)** as shown in **Figure 7**. Remove the locking element and restore the screw connection.

⚠ If the ceiling runs parallel to the floor, the locking element must not be removed!

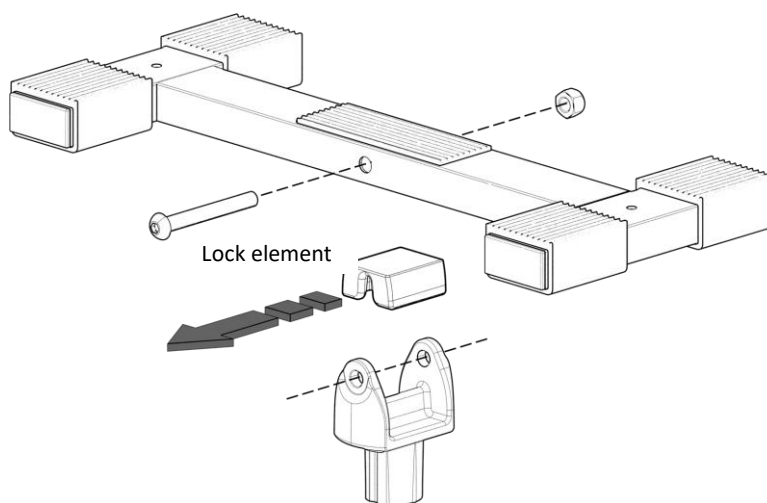**Figure 7**

## Instruction manual

### Floor-ceiling bar "Mister X"

Article: MRX-7621-xx

4.7 When mounted on a sloping ceiling (sloping roof) or at a room height greater than 305 cm using the MRX-7625-xx extension tube available as an accessory, the floor and ceiling bar must be attached to the installation site by screwing. The same applies if a stable and backlash-free installation is not possible.

The attachment of the floor and ceiling bar must be done by screwing it to the positions shown in Figure 8.

In the case of a ceiling and / or floor construction made of wood, they can possibly use the supplied **wooden screws (K)**. Check the screw length for suitability before assembly. Make sure that the screw connection is made in the beams.

⚠ At Cover- and soil construction made of other materials (e.g., concrete) is accordingly suitable mounting material to Use (not included).

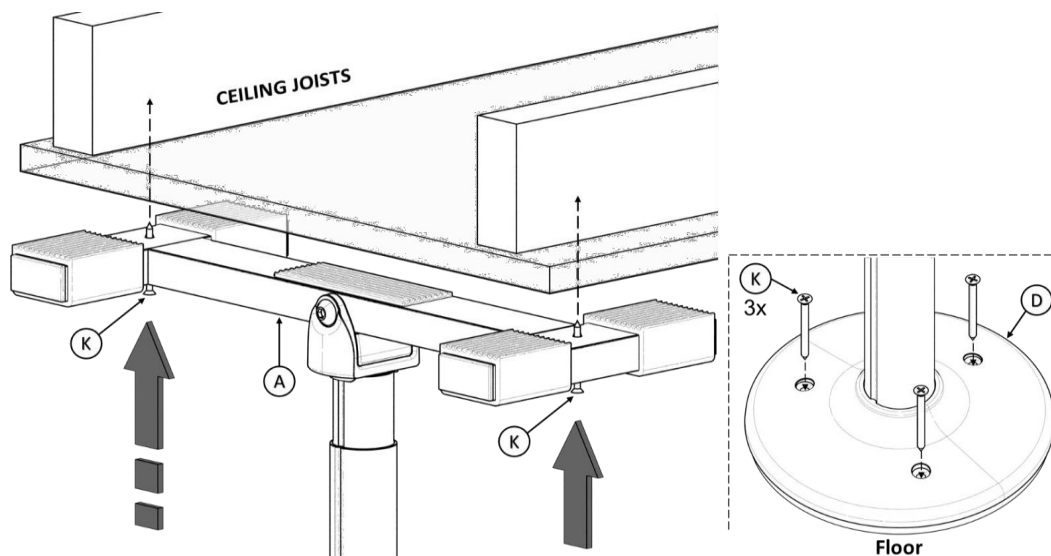
Figure 8

### 5.0 Cleaning instructions:

Clean the floor ceiling bar "Mister X" regularly with a damp sponge, mild soap and a household disinfectant. Dry it with a soft cloth. Do not use abrasive detergents such as scouring milk or powder, this can lead to destruction or damage to the surfaces and threads.

### 6.0 Reuse / Lifetime:

Due to its simple and solid construction, the floor ceiling bar "Mister X" is suitable for reuse. The number of re-inserts depends on the state of use of the device. Before each reuse, the floor ceiling bar "Mister X" must be hygienically prepared and the technical and functional safety of the product checked and, if necessary, restored. The assembly must be carried out as described in point 4.

If the product is used as intended in accordance with the specifications, the service life is expected to be 4 years. However, the effective service life can vary depending on the frequency and intensity of use as well as the care and general condition.

### 7.0 Technical data:

Suitable ceiling height:	213 – 305 cm to 370 cm with the extension tube MRX-7625-xx (accessories)
User weight:	max. 136 kg
Weight of the product:	9.3 kg
Material:	Aluminum tube anodized, Aluminum die casting powder coated, Screws made of stainless steel, Plastics

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

**8.0 Precautions / Warnings:**

**Please read the following instructions carefully before using the product.**

- ⚠ Check regularly that the floor ceiling bar "Mister X" is securely fastened between floor and ceiling.
- ⚠ Additional safety measures should be taken for persons belonging to certain risk groups. This includes persons with a mental impairment (including persons under the influence of medication), very large persons or persons with severe obesity, as well as any condition that impairs a person's ability to use the floor-ceiling bar "Mister X" without assistance.
- ⚠ If you have an illness or injury, or are taking medications that affect your sense of balance or your ability to stand or sit, please consult your physician before using this product. Improper use of the product can lead to serious injury or damage.
- ⚠ The product is not intended to carry the full weight of a person. This product is only used as a balance aid and support when standing, standing up and sitting down.
- ⚠ The instructions associated with the product must be followed carefully to ensure that the product is properly installed and assembled. Failure to follow the instructions may result in serious injury to the user(s) of the product.
- ⚠ Make sure that children do not play with the product or get attached to it.
- Disclaimer: Neither Standers, Inc. nor the manufacturer/distributor shall be liable for any damage or injury that may or may arise from improper use or assembly of the product.
- In case of installation problems, please contact the sales office (specialist trade) or directly to  
rehasstage GmbH  
Artlandstraße 80  
49610 Quakenbrück / Germany  
[gm@rehasstage.de](mailto:gm@rehasstage.de)
- In the event of serious undesirable incidents (deterioration of the user's condition) caused by the use of our product, we are obliged to report this to the Federal Institute for Drugs and Medical Devices. Therefore, we ask you to inform us directly in such a case:  
rehasstage GmbH  
Artlandstraße 80  
49610 Quakenbrück / Germany  
[gm@rehasstage.de](mailto:gm@rehasstage.de)

**9.0 Disposal / recycling of materials:**

If the tool was provided to you free of charge, it may not be yours. If it is no longer needed, please follow the instructions you received from the organization that provided you with the tool regarding the return. The following section describes the materials used for the aid, with information on how to dispose of or recycle the aid and how to package it. Special regulations regarding disposal or recycling may also apply on site, which must be taken into account during disposal.

- Aluminum: Tubes and brackets
- Stainless steel: clamping device, fastening screws
- Plastic: threaded protective cover, anti-slip rubbers, bag for mounting material
- Carton: Outer packaging

Disposal should be carried out by a disposal company or a public disposal point.

**10.0 Warranty:**

In addition to the statutory warranty, rehasstage offers a warranty on this product against manufacturing or material defects of 2 years from the date of purchase. The warranty only applies to the first user of the item. Keep the invoice well, which is your proof of purchase. The warranty does not cover defects caused by wear and tear, improper handling or commercial use of the device. The warranty expires if the item is repaired or structurally altered by anyone other than rehasstage itself or a dealer authorized by rehasstage. The warranty obligation expires if repairs or changes or replacement of individual parts that are not original rehasstage spare parts are carried out by parties other than the seller or the

**Instruction manual**  
**Floor-ceiling bar "Mister X"**  
**Article: MRX-7621-xx**

supplying company. During the warranty period, rehastage is responsible for deciding to repair the device free of charge or to replace it free of charge.

**11.0 Explanation of symbols used, markings and information to be provided:**

				
Manufacturer, name and address of the manufacturer	Authorised representative of the manufacturer within the European Community	Medical	Attention! Information such as warnings or precautions	Follow the instructions for use!

Subject to technical changes

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### Für das Bett

FRIDA

ADV-Traveler

Eco-Rail

Bed-Cane

IBT-Bettgriff

Mobi-Rail

### Für Überall

POLE

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HUGO

### Für das Auto

Metro

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Uplift